


Quick Start Set-up:

- Remove system from the box by lifting the top handles. Place on sturdy table. Remove the plastic covering from the A3500 system. Remove accessory box, funnel and power cord and store box for future use.
- Take a moment to review the A3500 photo and Parts Description.
- Choose the disposable plastic tip size (M or L) which best suits the treatment to be performed, e.g. medium tip for face, large tip for chest/back. Place the plastic tip on the top of the handpiece using a twisting motion while applying downward pressure until the pin on the handpiece fits into the groove on the plastic tip.
- Fill the clean container (the one without the filter) with crystals to the Full Line. Crystals should not be below the 450 mark on the container.
- Turn on fan switch located in the back left side of the system.
- Turn on system switch on front panel.
- Please refer to Operator Manual for treatment procedure.

A3500 PowerPeel Microdermabrasion System

Parts Description (next page)

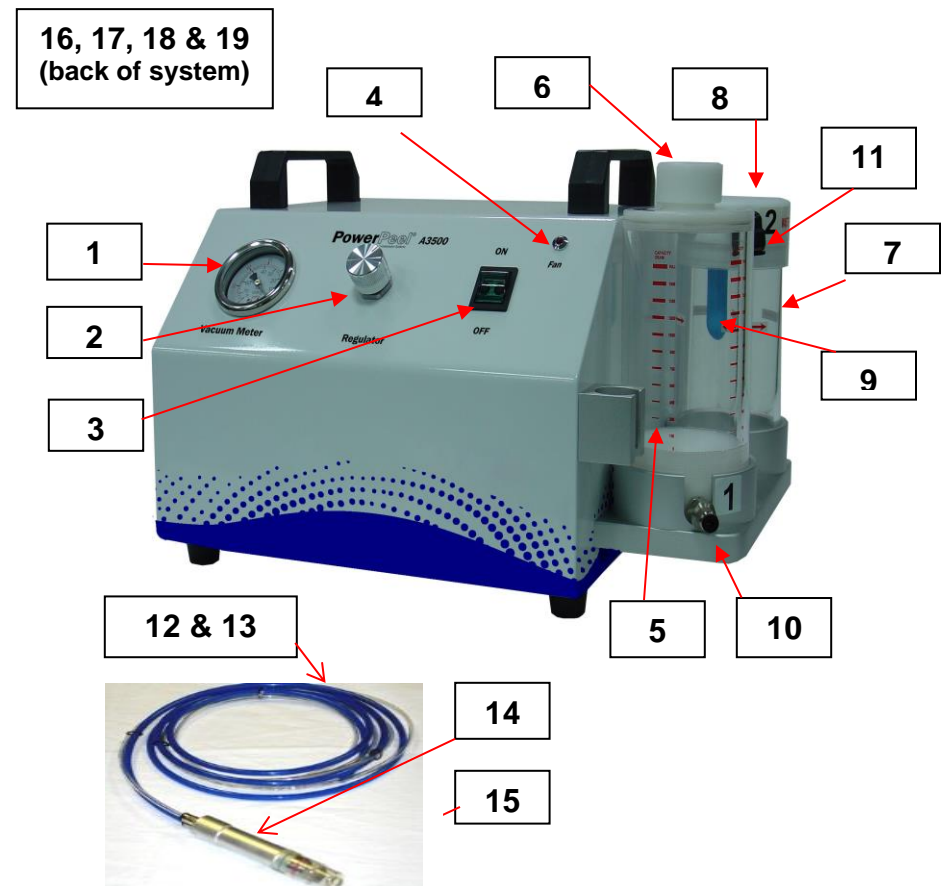

A3500 PowerPeel Microdermabrasion System

Parts Description

- | | | |
|-----|-----------------------------------|---|
| 1) | Vacuum Meter | Indicates the power setting for the vacuum flow (suction). |
| 2) | Regulator Knob | This regulates the power setting for the vacuum flow (suction). |
| 3) | System Switch | Located on the front of the A3500. |
| 4) | Fan Light | Light indicating fan is on. |
| 5) | Clean Crystal Container | Houses the clean crystals. |
| 6) | Clean Container Cap | Clean Container Cap is easily removed by lifting up on the cap to add crystal to the container. Place cap back onto container and push down to secure seal. |
| 7) | Waste Crystal Container | Houses the used (waste) crystals. |
| 8) | Waste Container Lid | Waste Container Lid is easily removed by lifting up on the cap to dispose of used crystals. Place lid back onto the container and push down to secure seal. |
| 9) | Waste Filter (blue) | Stops the crystals from getting into the vacuum pump. Waste filter is located in the waste container. |
| 10) | Fitting for Crystal Delivery Tube | Fitting where Crystal Delivery Tube is connected. |
| 11) | Elbow for Crystal Recovery Tube | Elbow where Crystal Recovery Tube is connected. |
| 12) | Crystal Delivery Tube (small) | One end of the tubing connects to the fitting and the other end connects to the handpiece. |
| 13) | Crystal Recovery Tube (large) | One end of the tubing connects to the elbow and the other end connects to the handpiece. |
| 14) | Operator Handpiece | Applicator that vacuums crystals across the surface of the skin. |
| 15) | Disposal Plastic Tip | A plastic tip that is placed at the end of the handpiece to direct crystal flow. |
| 16) | Fan Switch | Located in left rear corner of system. Turn the switch to the "on" position when starting the microdermabrasion system. This acts as a cooling system for the machine (not visible in photo). |
| 17) | Power Cord Plug | Located in left rear corner of system. Plugs into 120 Volt wall outlet (not visible in photo). |
| 18) | Fuse Housing | Located in center rear of system. Contains two 5 AMP fuses (not visible in photo). |
| 19) | Serial Number Plate | Manufacturer's identifying serial number located in center rear of cabinet (not visible in photo). |

To order the above parts, call UBS Aesthetics at 1-800-925-5022.